



Ultra Light Drape™

Instructions for Use



Step 1
Open light drape using sterile technique.
Ensure Drape is inside out-tabs are inside
drape



Step 2
Attach the blue universal handle
over surgical light handle.




Step 3
Hands of positioner remain
inside of light drape. Flip
drape over top of light lens.



Step 4
Hands of positioner remain
inside of light drape. Flip
drape over bottom of light lens.



Step 5
Pull any excess drape behind lens
and secure with sticky straps
across back of light

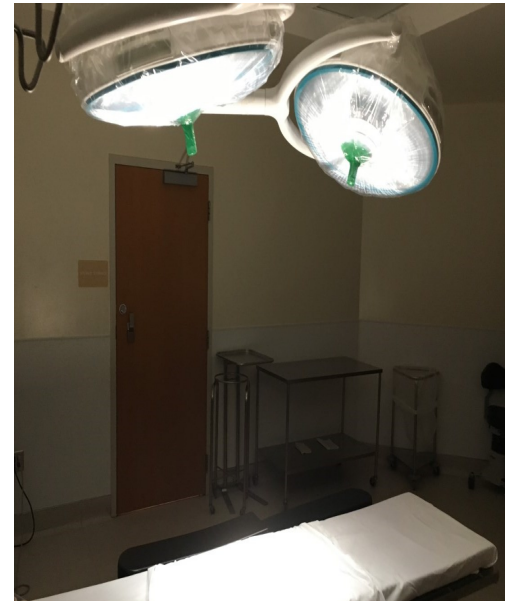

Step 6
Position Sterile Light over surgical field.
Remove drape after procedure.

Ultra Light Drape™



Protect the integrity of your Surgical Lights with the New Ultra Light Drape. The Ultra Light Drape is the only product available to defend the sterile field from above.



Caution: The disposable Surgical Light Drape is a single use item. Multiple uses of this product could result in infection or cross contamination. Discard after single use.

Drape is intended to provide sterile barrier protection for surgical procedures only. Staff must comply with hospital guidelines for maintaining and cleaning surgical light between procedures.

Ordering Information:

AM1005-Ultra Light Drape without handle 20/CS

AM1006-Ultra Light Handle with handle 20/CS



Manufactured for: American Surgical Specialties Company 475 Metroplex Dr Suite 401 Nashville, TN 37211

(O)1-866-387-7021 sales@americansurg.com

US Patented Product