



Ultra Light Drape

Instructions for Use



Step 1

Open light drape using sterile technique. Ensure Drape is turned inside out-tabs are inside drape



Step 2

Attach the blue universal handle over surgical light handle OR use your current disposable handle in the same fashion



Step 3

Hands of positioner remain inside of light drape. Flip drape over top of light lens.



Step 4

Hands of positioner remain inside of light drape. Flip drape over bottom of light lens.



Step 5

Pull any excess drape behind lens and secure with sticky straps across back of light



Step 6

Position Sterile Light over surgical field for procedure



Now your surgical Light can be manipulated without inadvertently touching the lens or debris falling into the surgical wound or the sterile field



After surgical procedure, remove drape and discard according to hospital waste and environmental policy

AM2005-Large

20 Units/CS

AM2006-Small

20 Units/CS

AM2007-Small /Handle



Caution: The disposable Surgical Light Drape is a single use item. Multiple uses of this product could result in infection or cross contamination. Discard after single use.



Drape is intended to provide sterile barrier protection for surgical procedures only. Staff must comply with hospital guidelines for maintaining and cleaning surgical light between procedures.

